# LuViva<sup>®</sup> Advanced Cervical Scan

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The screenshot shows the LuViva Scan result screen. At the top left is the LuViva logo with the tagline "Advanced Cervical Scan". At the top right are "Home" and "Help" buttons. Below the header, a "Cytology Selected:" dropdown menu shows "ASC-US". The main content area is split into two columns: "Referral Criteria" and "LuViva Result". Under "Referral Criteria" is "ASC-US". Under "LuViva Result" is a yellow box containing "Moderate ()". Below this is a horizontal bar with three segments: a green segment labeled "Low", a yellow segment labeled "Moderate", and a red segment labeled "High". At the bottom of the main area are two buttons: "Change Cytology" and "Image Review". At the very bottom is a progress bar with steps 1 through 10, where step 10 is labeled "Result".

LuViva  
Advanced Cervical Scan

Home Help

Cytology Selected: ASC-US

**Referral Criteria**      **LuViva Result**

ASC-US      Moderate ()

Low      Moderate      High

Change Cytology      Image Review

Step 1 Step 2 Step 3 Step 4 Step 5 Step 6 Step 7 Step 8 Step 9 Step 10 Result

The screenshot displays the LuViva Scan software interface. At the top left is the logo for LuViva Advanced Cervical Scan. At the top right are navigation buttons for Home and Help. Below the logo is a blue bar with the text "Cytology Selected: LSIL". The main area contains two images: a full view of a cervix on the left and a magnified view of a specific area on the right. A dashed white box on the left image indicates the area being magnified. To the left of the images is a vertical slider with a gear icon at the top and a sun icon at the bottom, used for adjusting the magnification. Below the left image is the instruction "Touch the left picture to position the magnifier." Below the right image is the instruction "Touch magnified image to center." To the right of the images are two buttons: a green button with a plus sign labeled "+ Add Image to Report" and a grey button with a red X labeled "Remove Image from Report". A speech bubble points to the "Add Image to Report" button with the text "Click here to add the image on the right to the report." At the bottom of the interface are four buttons: "Back" (with a blue arrow), "View Report" (with a green checkmark), "Save Report" (with a blue floppy disk icon), and "Exit Test" (with a green checkmark).

LuViva  
Advanced Cervical Scan

Home Help

Cytology Selected: LSIL

This is the area that is enlarged on the right.

Touch the left picture to position the magnifier.

Touch magnified image to center.

+ Add Image to Report

Remove Image from Report

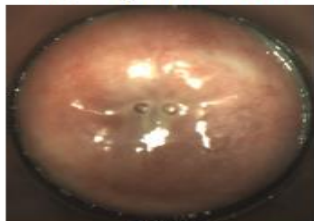
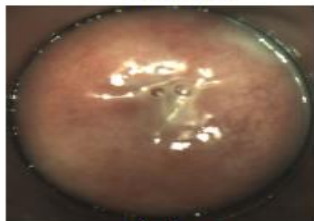
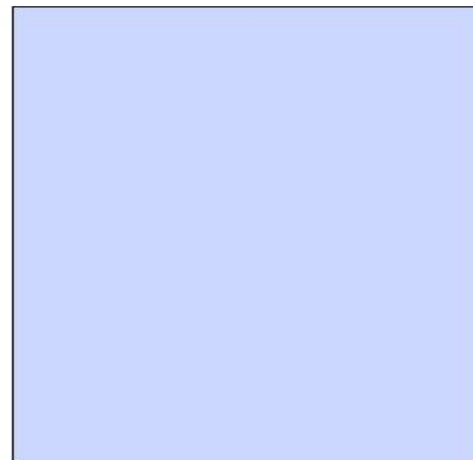
Back View Report Save Report Exit Test

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